Appendix C : Respirator Training Checklist

Review selection process of respirator	
Inspection of respirators	
Valves Rubber/neoprene Straps & fasteners Cleanliness Filter assembly Filter changes and expiration dates Proper storage of respirators	
Discard faulty respirators or replace defective parts	
Issue new respirators to those that need one	
Review straps and adjustment capabilities	
Discuss comfort concerns	
Review filter capabilities	
Review change-out schedule or end of service life in	dicators
Don Respirator	
Perform User seal check procedures	
Review need to do this check every time respirator is	s worn
Make adjustments if necessary	
Fit test using isoamyl acetate or stannic chloride whi	le standing and:
 □ Breathing normally □ Breathing deeply □ Turning head from side to side □ Move head up and down – inhale in up position □ Talking - read Rainbow passage, count backward from □ Bending over or jogging in place □ Breathing normally (Each test exercise shall be performed for one minute) 	m 100, or recite something
Fill out paperwork Question and answer period File forms in employee's folder	
Signature of Trainer:	Date:
Signature of Trainee:	Date: